

GL Dental Chair - Inova New Line Dental Set

Procedures for selecting the LED / Lamp Dental Light and configuring the Right / Left foot control

Selection of the Dental Lights - LED / Lamp.

NOTE: The board leaves the factory \prepared for the Lamp Dental Light.

From Lamp to LED:


- Turn off the main switch of the dental chair;
- Locate on the electronic board the jumper JP1 and close the connection, joining the two pins (jumper closed);
- With the activation button of the dental light (02) pressed, turn on the main switch of the dental chair. Maintain the button (02) pressed;
- The dental chair will emit one long Beep and a sequence of short Beeps;
- With the dental chair Beeping, release the activation button of the dental light (02);
- Turn off the main switch;
- Remove the Jumper JP1 positioning it as at the start (jumper open).

From LED to Lamp:

- Apply the same procedure using the activation button of the dental light (01).

Note:

- This calibration is exclusive for the dental light not activating any movements of the dental chair.

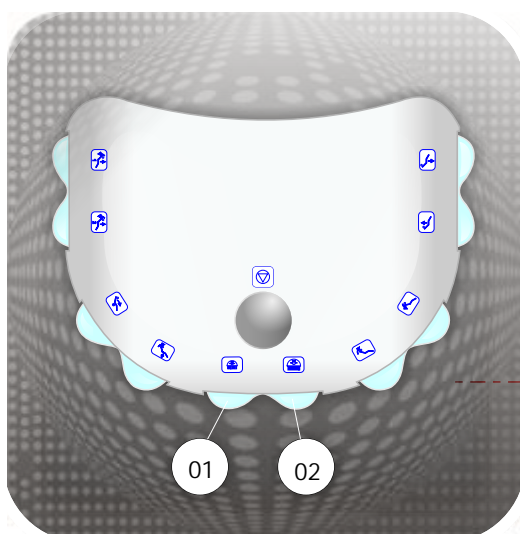


Jumper JP1
open



Jumper JP1
closed



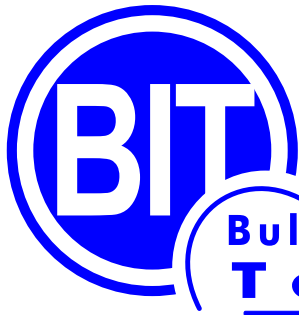

GL Dental Chair - Inova New Line Dental Set

Procedures for selecting the LED / Lamp Dental Light and configuring the Right / Left foot control

Configuration of the foot control – Right to left

From right to left:


- Turn off the main switch of the dental chair;
- Locate on the electronic board the jumper JP1 and close the connection, joining the two pins (jumper closed);
- With the button (03) figure 01 pressed, turn on the main switch of the dental chair. Maintain the button (03) pressed;
- The dental chair will emit one long Beep and a sequence of short Beeps;
- With the dental chair Beeping, release the button 03 figure 01;
- Turn off the main switch;
- Remove the Jumper JP1 positioning it as at the start (jumper open).
- After configuring the foot control, stick the indicator adhesive labels on the foot control of the dental chair (as per figure 02).

From left to right:

- Apply the same procedure using the button (04) figure 2
- After configuring the foot control, stick the indicator adhesive labels on the foot control of the dental chair (as per figure 01).

Note:

- This calibration is exclusive for the foot control not activating any movements of the dental chair.

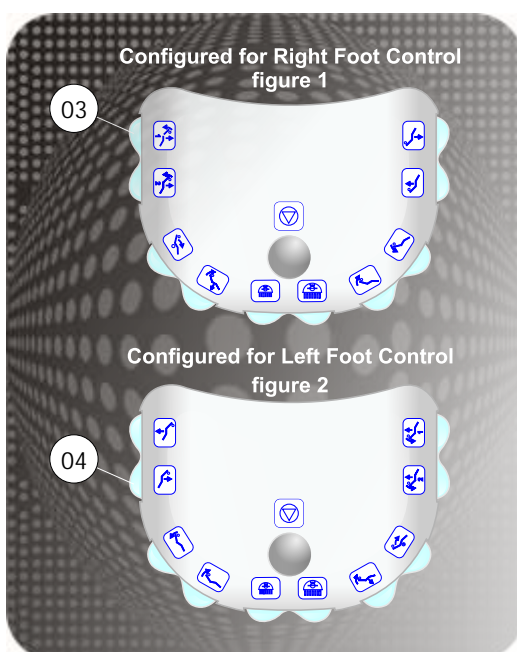
 Jumper JP1
open

 Jumper JP1
closed
