

"RECORDER OUT OF PAPER"	No paper is in the recorder.	Place the paper into the recorder.
"RECORDER PAPER JAM"	The paper in the recorder is jammed.	Place the recorder correctly and try again.
"RECORDER COMM ERR"	The communication of the recorder is abnormal.	In the recorder setup menu, execute the function of clearing record task. The function can make the host and the recorder connect again. If the failure still exists, contact the manufacturer for repair.
"RECORDER S. COMM ERR"		
"RECORDER PAPER W.P."	The paper roll of the recorder is not placed in the correction position.	Place the paper roll in the correct position.
"REC NOT AVAILABLE"	Cannot communicate with the recorder.	In the recorder setup menu, execute the function of clearing record task. The function can make the host and the recorder connect again. If the failure still exists, contact the manufacturer for repair.
"NIBP INIT ERR"	NIBP initialization error	Execute the reset program in the NIBP menu. If the failure still exists, contact the manufacturer for repair.
"NIBP SELFTEST ERR"		
"NIBP ILLEGALLY RESET"	During NIBP measurement, illegal reset occurs.	Check the airway of NIBP to see if there are clogs. Then measure again, if the failure still exists, contact the manufacturer for repair.
"NIBP COMM ERR"	The NIBP communication part has problem.	Execute the reset program in the NIBP menu. If the failure still exists, contact the manufacturer for repair.
"LOOSE CUFF"	The NIBP cuff is not connected correctly.	Re-connect the NIBP cuff.
"AIR LEAK"	The NIBP cuff is not connected correctly or there are leaks in the airway.	Check the connection of each part or replace with a new cuff. If the failure still exists, contact the manufacturer for repair.
"AIR PRESSURE ERROR"	Problem happens when measuring the curve. The system cannot perform	Check the connection of each part or replace with a new cuff. If the failure still exists, contact the