

Warning message	Cause / Remedy
<p style="text-align: center;">Error 1: Vacuum system</p>	<p>The monitoring time for reaching the evacuation pressure for the individual pressure cycles, pressure release, or for reaching the minimum drying pressure has been exceeded:</p> <ul style="list-style-type: none"> • Check that the door seal and the lip of the opening to the pressure chamber are intact and clean. • Check that the autoclave is standing properly (see Section: 2.3) • Check that the outflow of condensate is not obstructed by fallen instruments, pieces of filter paper, etc. on the floor of the pressure chamber. • Check for leaks using the "Vacuum test" program. <p>If this occurs repeatedly, inform your specialist dealer.</p>
<p style="text-align: center;">Error 2: Steam generator</p>	<p>The monitoring time for a heating phase or for reaching the sterilization pressure has been exceeded:</p> <ul style="list-style-type: none"> • maximum load exceeded • voltage of mains power supply is too low; check mains supply; try autoclave on different power circuit. <p>If this occurs repeatedly, inform your specialist dealer..</p>
<p style="text-align: center;">Error 4: Pressure release</p>	<p>The monitoring time for pressure release was exceeded:</p> <ul style="list-style-type: none"> • Check that the cooling water outflow allows steady drainage, without kinks (see Section: 2.6) <p>If this occurs repeatedly, inform your specialist dealer..</p>
<p style="text-align: center;">Error 6: Ventilation</p>	<p>The monitoring time for the ventilation of the pressure chamber was exceeded.</p> <ul style="list-style-type: none"> • The sterile filter is clogged, there will have been an earlier warning message (see Section: 7.3) • Exchange filter.
<p style="text-align: center;">Error 8: Timebase</p>	<p>Maximum difference between the program duration and the internal clock exceeded:</p> <ul style="list-style-type: none"> • If this occurs repeatedly, inform your specialist dealer.
<p style="text-align: center;">Error 9: Door open</p>	<p>Door not closed properly</p> <ul style="list-style-type: none"> • Press grip down until contact is made (display should then show "Door closed")! <p>If this occurs repeatedly, inform your specialist dealer.</p>
<p style="text-align: center;">Error 10: Overh steam gener.</p>	<p>The capillary tube level regulator is open at the start of the program (error report immediately after start), or the monitoring time until refilling with demineralised or distilled water during the program (until the end of sterilization) is exceeded:</p> <ul style="list-style-type: none"> • This problem can arise because after stopping a program and immediately restarting - wait for two minutes and try starting again. <p>If this occurs repeatedly, inform your specialist dealer.</p>
<p style="text-align: center;">Error 12 Door locking</p>	<p>Maximum permissible time for door locking:</p> <ul style="list-style-type: none"> • Check the locking bolt can move freely <p>If this occurs repeatedly, inform your specialist dealer.</p>
<p style="text-align: center;">Error 13: No cooling water</p>	<p>The cooling water pressure switch has closed during the program (see message "Warning no cooling water" - page 31).</p>