

7. Error Codes

This function is currently not available.

Note: Error Code E:01 will be displayed after switching on the eBike, when the cuff was moved during self-test. In this case the amplifier cannot be initialized and the eBike has to be started again. E:01 will not be recorded and will not appear here.

Note: If the error code E:01 appears again, even though the cuff was not moved, the BP module is probably defective. To be able to run an exercise test without the BP module, disable the blood pressure DIP switch (see Figure C-6 DIP Switch and Jumper Settings).

Error Code E: 02 = MRD, offset or gain error

If a module version is detected that is not compatible with the installed software package, the boot procedure will be aborted with one of the following error codes:

Wrong software in **IFP** board: the boot procedure is aborted and the error code **E:90** displays.

Wrong software in **AZP** board: the boot procedure is aborted and the error code **E:91** displays.

Wrong software in **NIBP** board: the boot procedure is aborted and the error code **E:92** displays.

Wrong software in **MRD** board: the boot procedure is aborted and the error code **E:93** displays.

If the **NIBP** software does not work correctly during the regular use of the ergometer, the error code **E:10** displays.

If the **MRD** software does not work correctly during the regular use of the ergometer, the error code **E:20** displays.

Note: During the boot procedure, the software is checked in the above order. The procedure is aborted, when the first incompatible software is detected. Only the first detected mismatch is indicated, i.e., the remaining modules are not checked.

Remedy: Install software FW1.8 again as described in section 6.

