


5.5 Connecting a Footswitch


The footswitch can control “Freeze” operation and “Print” operation remotely.

Connect the footswitch connector to the port at the bottom of the ECG panel, as shown in the figure below, and then rotate the connector to fix it in position.

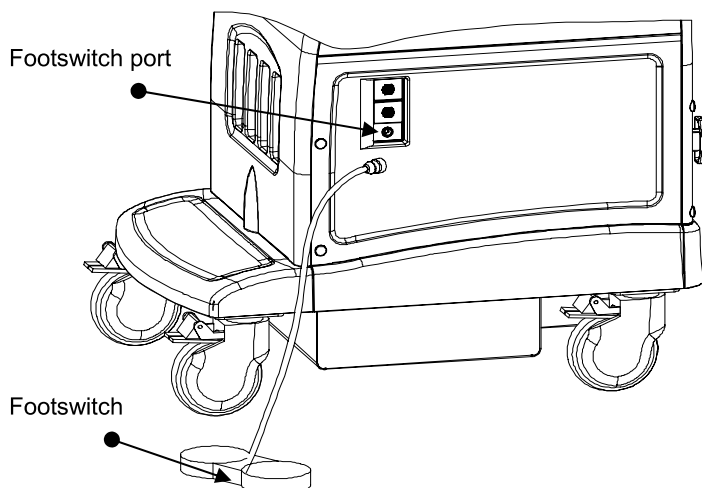

5.6 Connecting a graph/text printer

As shown in the figure below, a graph/text printer has a power cable and data cable, and the power cable shall be directly connected to a wall receptacle as required. The printer model shall be a specified model supported by this system. Refer to “Recommended Peripheral Devices” for specific models.

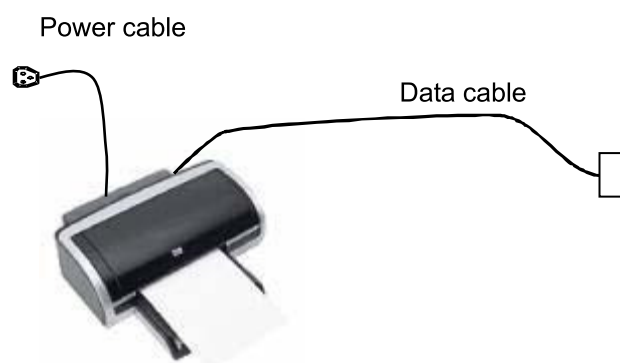

The procedures are described as follows:

- (1) Connect the data cable to USB port of the system.
- (2) Connect the power cable to a wall receptacle.
- (3) Turn on the power of the ultrasound system.
- (4) The installation process is complete.