



## Anesthesia Delivery & Ventilation

Beech House, Chiltern Court Asheridge Road, Chesham,  
Bucks HP5 2PY Telephone: 01494 784422 Fax: 01494 791497

### FRONTLINE ANNUAL SERVICE KIT (ALL MODELS) PART NUMBER 940444-1

Kit contains all parts required for annual service on a top specification machine. When servicing a lower specification machine some parts will be surplus to requirements and should be discarded.

#### **FITTING INSTRUCTIONS.**

Remove cylinders from yokes and replace Bodoc seals on yokes.

Remove pipelines from NIST fittings and replace 'O' rings on pipelines and replace filter in NIST connector on machine.

Unscrew Dzus spring plates from back bar. Replace Dzus springs and refit plates.

Replace 'O' rings on "Selectatec" type back bars.

Remove rotameter knobs from flowmeter. Release flowmeter cover from machine by removing the three screws at top rear of flowmeter.

Remove each flow tube in turn and replace all 'O' rings on tube mountings and filter in lower insert. Using a suitable brush and cleaner, clean and replace flow tube, ensuring correct position and orientation.

When all tubes have been serviced, replace flowmeter cover and refit knobs.

Test to service schedule.

#### **NOTE: Genius/Focus Annual kit (940464)**

Eight AA batteries are incorporated into this Service Kit for replacement within the O2 Monitor.