

6.3.2 Change piston/cylinder system

To change the piston-cylinder system the following work should be carried out in the given order:


Select on the touch screen:

Menu -> Functions -> Actions -> Change dosage

Moves the dosing agent piston into a position where the change is possible.


- Switch off the analyser
- Open the machine door
- Switch off the piston-cylinder system from the valve block
- Take the piston-cylinder system out through the flaps and pull out
- Insert the piston-cylinder system in the reverse order
- Close the analyser door

6.3.3 Change the sample cannula and the rinse vessel

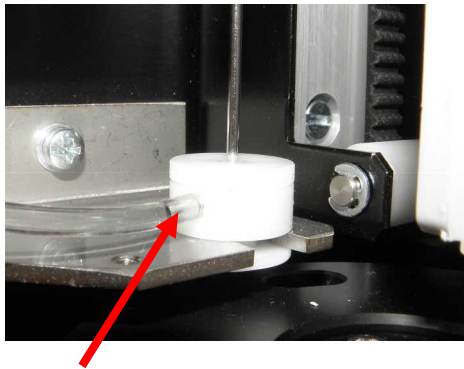

1

Fig. 6.1 View of rinsing cup

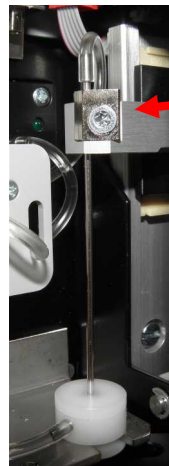

2

Fig. 6.2 View of sample cannula

Extension

1. Switch off the analyser
2. Take the suction tube from the containerkit
3. Place the suction end at the level of the table in a small cup or on a little cellulose wadding (a little fluid will run out).
4. Pull off the tube (1)
5. Release the screw (2)
6. Remove the needle
7. Take the rinse vessel out of the holder