



User Defined Key

The **User Defined Key** menu allows you to specify user-defined keys. To assign functions onto numeric keys (0-9), select the **Enable Hot Key** check box and assign a key function. The following table shows the functions that you can assign for available keys.

Table 9-3 User-defined key

Available keys	Function
User Key 1	*Power Preset
User Key 2	Biopsy Guide
#0	Spatial Compound
#1	SRI
#2	Auto Calc On/Off
#3	Virtual Convex Image
#4	Full D
#5	Full M
#6	U/D Flip
#7	L/R Flip
#8	Rec/Pause
#9	Clear All
	Transducer Port 1~3
	Quick ID
	Initial
	Eject
	DVR
	End Exam
	Quad view



NOTE

***Power Preset** allows you to quickly change the transducer and its preset.