
## DIAGNOSTIC MEDICAL SYSTEMS & SERVICE

## iU22 Power Supply Test Points PN: 4535-6119-5282

Verification is a very important part of diagnosis when trying to solve any type of problem. Take for instance, a Technologist says "The machine just powered down on its own", or "It works fine until I try to use the Doppler, then it

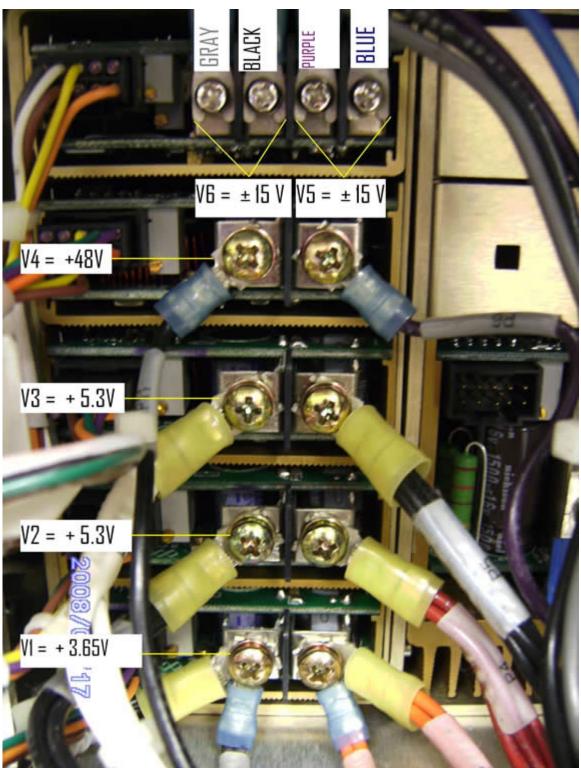

locks up". As technician, you first need to verify that the symptoms reported are accurate. If they are, first instinct is to head straight to the power supply. Our technical staff support has provided system voltage test points (all voltages must be within 2% to 5% tolerance), each supply has a potentiometer to adjust voltage within its tolerance limit. If your power supply is not adjustable, please call our support staff for technical assistance and we'll help you be the hero of the day.

Good Luck and Be Safe.

Compliments of: Midwest Imaging 920 Hemsath Road Suite 102 St. Charles, MO 63303

877.889.8223 North America 636.925.0030 Worldwide 636.925.0070 Fax www.mwimaging.com info@mwimaging.com