

25	10000
30	8037
35	6500
INCUBATOR HEAT SINK TEMPERATURE SENSOR THERMISTOR	
Thermistor	Control bd J3 pins 1-2
Temperature (C)	Resistance (Ohms)
20	25000
25	20000
30	16102
35	13048
40	10636
80	2506
90	1827
100	1353
110	1017
120	775

E-BASE MOTOR

If you must replace a footswitch when the bed is all the way down, disconnect the shorted footswitch, the short switch pins on the relay board. On units with 1.5 software or higher, use pedal screen on service screen to raise or lower the bed.

The e-base is driven at 24 volts DC (acceptable range 24-32). The following chart shows the control si