

5. Supplement of Consumed Expendables

Order the following expendables with numbers shown in the table.

	Order No.
Chin Rest Paper	K9L-TB45 #102
Chin Rest Pin	K9L-TB45 #101
Printer Paper	STH-148
Fuse	21301.6 (T1.6A L250V)

6. Periodical Check

To use the KT-800 safely for long time, we recommend you to check it once a year.

Contact your Kowa dealer where you purchased this KT-800 for contents of check and fees.

7. Calibration and Inspection (items that requires inspection by qualified service personnel)

This is a precision measuring instrument. Please have qualified service personnel calibrate the instrument at least once every 2 years.

4 types of model eyes - 10, 20, 30, 40mmHg - are used to calibrate.

For service fee, please contact Kowa representative listed in the back cover.

8. Repair of Equipment

Whenever the KT-800 must be returned to the manufacturer for repair or maintenance, contact your kowa dealer.

9. Error messages

This section describes error messages that appear when a device malfunction occurs.

Message	Description	Remedy
ERROR SRAM ACCESS	Displayed when the data in SRAM has been damaged	Turn off the power supply and contact Kowa representative listed in the back cover.
ERROR EEPROM ACCESS	Displayed when the data in EEPROM has been damaged	Turn off the power supply and contact Kowa representative listed in the back cover.
ERROR MEMORY ACCESS	Displayed when the data in SRAM and EEPROM has been damaged	Turn off the power supply and contact Kowa representative listed in the back cover.
ERROR CPU ACCESS	Displayed when there are no replies from the CPU	Turn off the power supply and contact Kowa representative listed in the back cover.
ERROR PRINTER OFFLINE	Displayed when the printer is not recognized at the time of executing printing	Turn off the power supply and contact Kowa representative listed in the back cover.
PAPER END	Displayed when there is no printer paper or the printer cover is open	Supply the printer paper or close the printer cover.
CAUTION	Displayed when the electrical current protective circuit for upper/lower motor or chin-rest motor is activated	Auto-release will be activated in 3 seconds. If the incident occurs frequently, turn off the power supply and contact Kowa representative listed in the back cover.