

Anleitung

Re-Initialisierung Inkubator Monitor
Thermocare Vita

Deutsch

Bis Monitorsoftware Version 2.70:

Inkubator einschalten und „Taste 1“ drücken und gedrückt halten, bis Temperatur- und Trendanzeige auf dem Display erscheint.

Überprüfung:

Alle Uhrzeiten unterhalb der Zeitachse in der Trendanzeige müssen auf 00:00 stehen.

Ab Monitorsoftware Version 2.71:

Inkubator einschalten und „Taste 1“ gedrückt halten, bis intermittierender Ton ertönt, dann „Taste 1“ los lassen, Drehknopf drücken und halten, bis Temperatur- und Trendanzeige auf dem Display erscheint.

Überprüfung:

Alle Uhrzeiten unterhalb der Zeitachse in der Trendanzeige müssen auf 00:00 stehen.

English

Up to monitor software version 2.70:

Switch incubator on and press and hold “Key 1” until temperature and trend display is shown.

Check up:

All clock times under the time axis must be 00:00.

Since monitor software version 2.71:

Switch incubator on and press and hold “Key 1” until intermittend alarm sounds. Now release “Key 1” and press and hold rotary knob until temperature and trend display is shown.

Check up:

All clock times under the time axis must be 00:00.

