

QA/Expert

SPIKE-RESONANT Check

[Log-File](#)

Measurement: Some sequences are silent and can take 10 minutes


Image: Raw Data

Image: file:/C:/MedCom/service/html...Expert/SfpSpikeCheckExpert/gfx/qa_ -1.ima

Window: 54 Center: 8

Message:

Image Display:

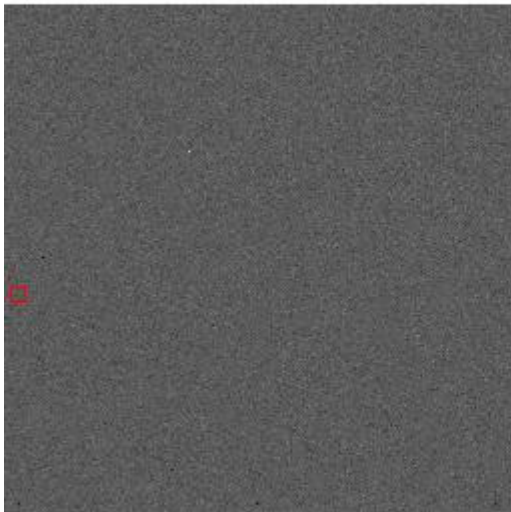

SeriesLoid Local 2.0.50725385
ImageNumber 1
256 * 256 (Lines * Columns)
FoV 500.0 * 500.0 (width * heigth)
TR 234.00 TE 1.00
Transversal
Patient position HFS
sequence spike_resonant

Spikes found in raw data

	Value	High Spec
Number of spikes found	10	0
Maximum spike amplitude	36.241	14.539


Image: RF Interference

Image: file://C:/MedCom/service/html...Expert/SfpSpikeCheckExpert/gf/qa_2-2.ima

Window: 137 Center: 68

Message:

Image Display:

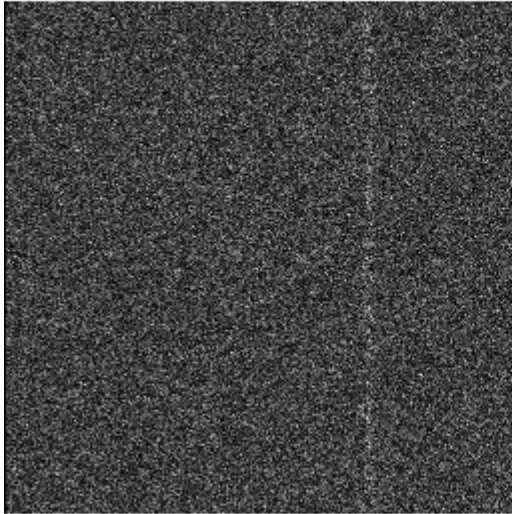

SeriesLoid Local 2.0.50725385
ImageNumber 1
256 * 256 (Lines * Columns)
FoV 500.0 * 500.0 (width * heigth)
TR 234.00 TE 1.00
Transversal
Patient position HFS

sequence spike_resonant

No RF Interference found in image!

Results are not in specification

Program finished successfully